

Florida Department of Law Enforcement

Alcohol Testing Program

OPERATIONAL PROCEDURES – INTOXILYZER 8000

1. All results are reported to three decimal places in g/210L. The instrument must display READY MODE prior to beginning the breath test. Push the START TEST button to begin the breath test:
 - Enter Breath Test Operator last name, first name and middle initial at USER prompts.
 - DATE/TIME. Verify the displayed date and time. Change if necessary. Press ENTER.
 - LAST AGENCY INSPECTION DATE. Verify the agency inspection date displayed. Press ENTER.
 - CYLINDER LOT#. Verify the dry gas standard cylinder lot number. Change if necessary. Press ENTER.
 - EXPIRATION DATE. Verify the dry gas standard cylinder expiration date. Change if necessary. Press ENTER.
 - OBSERVATION PERIOD BEGAN. Enter the time the observation period began (at least 20 minutes).
 - SWIPE DL OR PRESS ENTER. Either swipe the subject's driver license or identification card or press ENTER. Enter applicable information as prompted.
 - DIAGNOSTICS CHECK. The result must be OK.
 - AIR BLANK. The result must be 0.000.
 - CONTROL TEST. The result must be between 0.075 and 0.085, inclusive.
 - AIR BLANK. The result must be 0.000.
 - PROVIDE SAMPLE NOW. Have the subject provide a breath sample into the instrument.
 - AIR BLANK. The result must be 0.000.
 - PLEASE WAIT. The instrument will countdown the time remaining for the wait period.
 - AIR BLANK. The result must be 0.000.
 - PROVIDE SAMPLE NOW. Have the subject provide a breath sample into the instrument.
 - AIR BLANK. The result must be 0.000.

Note: If there is no 0.020 g/210L agreement between first and second breath samples, the instrument will automatically request a third breath sample as follows:

 - PLEASE WAIT. The instrument will countdown the time remaining for the wait period.
 - AIR BLANK. The result must be 0.000.
 - PROVIDE SAMPLE NOW. Have the subject provide a breath sample into the instrument.
 - AIR BLANK. The result must be 0.000.
 - CONTROL TEST. The result must be between 0.075 and 0.085, inclusive.
 - AIR BLANK. The result must be 0.000.
 - DIAGNOSTICS CHECK. The result must be OK.
2. If an external printer is used, FDLE/ATP Form 38 – Breath Alcohol Test Affidavit - Intoxilyzer 8000 will be automatically printed containing all the results. If no external printer is used, a printout slip containing all the results will be automatically printed.
3. Complete FDLE/ATP Form 38 – Breath Alcohol Test Affidavit - Intoxilyzer 8000.